

Patient Information Leaflet



SCR-01

Complete information and Instructions for Use can be found at www.securacath.com



© 2021 by Interrad Medical, Inc. All rights reserved.
1216-009-rA (2021-11-09)

Download the SecurAcath® App



securAcath®

For the Life of the Line™

Your catheter is secured with the SecurAcath® subcutaneous securement device.

About the SecurAcath Device

- The SecurAcath has small, blunt, flexible securement feet beneath the skin which act to stabilize the catheter right where it enters the body.
- The SecurAcath is intended to be used for any patient with an applicable percutaneous catheter.
- The SecurAcath is designed to secure the catheter for the entire duration of therapy, reduce catheter movement or dislodgement, and prevent therapy interruption.
- SecurAcath does not need to be changed. Consider it part of the catheter.
- The SecurAcath will be removed when the catheter is removed.
- The SecurAcath is composed of Nitinol, Polypropylene, and Thermoplastic Elastomer.

Possible Complications/Side Effects

The potential exists for serious complications including the following:

- Bleeding
- Brachial Plexus Injury
- Catheter Erosion Through the Skin
- Catheter Related Sepsis
- Insertion Site Infection or Necrosis
- Hematoma
- Intolerance Reaction to Implanted Device
- Laceration or Perforation of Vessels or Viscus

Device Interactions with Equipment

- You can have an MRI with the SecurAcath in place. SecurAcath is MRI Conditional.
- Misalignment of the catheter within the SecurAcath may lead to occlusion and/or catheter failure. Healthcare providers should flush the catheter after SecurAcath placement.

Care and Maintenance

- Care and maintenance of your catheter site should be performed routinely according to your healthcare provider's instructions.
- Your healthcare provider should know about SecurAcath before doing a dressing change.
- Place semi-permeable transparent dressing on somewhat loosely. If the dressing is applied too tightly, it can cause discomfort.
- Transparent dressing should cover the insertion site, SecurAcath, and catheter suture wings. Only the extension tubes should be outside the dressing to prevent pulling or kinking of the catheter.
- If there is any pain or discomfort at the insertion site, contact your healthcare provider for assessment of the catheter, dressing, and SecurAcath. Changing the dressing may ease pain or discomfort. Ensure the SecurAcath is lying flat on the skin and has not shifted from its original position. When replacing the dressing, make sure it is not too tight.
- If the catheter or SecurAcath has migrated from original placement, contact your healthcare provider.

Any serious incident that occurs in relation to SecurAcath should be reported to Interrad Medical and the Therapeutic Goods Administration (tga.gov.au)