



securAcath®

24-Hour Clinical Information Line
1-800-225-0000

*Go to www.securacath.com for more information

Helpful Info for Community Care Providers

The SecurAcath Subcutaneous Anchor Securement System (SASS) is a revolutionary new method for catheter securement that does not require sutures or adhesives. The unique design of the SecurAcath secures right at the insertion site using small, flexible securement feet placed in the subcutaneous tissue just beneath the skin.

SecurAcath has many benefits* that make catheter care much easier and reduce catheter complications. The SecurAcath lasts the life of the line and can:

- Significantly reduce risk of catheter-related infections
- Dramatically decrease catheter dislodgement and migration
- Decrease catheter replacement costs
- Improve efficiency of dressing changes
- Prevent catheter associated skin injury (CASI)

Your agency may be seeing patients with SecurAcath. We wanted to provide you with some valuable training tools to inform your field staff that care and remove PICCs in the community. In addition to the information and links below, my team and I are available to provide live virtual in-services that include didactic, videos and hands-on demonstrations. Let me know if you are interested in setting these up. Clinician education and confidence is essential to SecurAcath's success.

We want to provide some training resources on SecurAcath and its benefits. Please be sure everyone on your team has our **24 hour support line: 800-225-0000**. We are standing by to provide just-in-time training, so please don't hesitate to call.

Key points any clinician caring for SecurAcath should know:

- Secures for the life of the line, from insertion until the catheter is removed.
- When cleaning the site during a dressing change, just lift SecurAcath like a hinge, clean 360 degrees around the site with disinfection agent, let dry completely, and apply dressing per protocol. There is no need to open the SecurAcath for cleaning.
- Compatible with all dressings and add-on disks.
- There are two removal techniques, split and fold. Video links below need to be watched prior to removal.
- The smartphone App is free and provides convenient access to training videos and a phone icon direct link to our 24 hour clinical info number.
- Patients should receive a patient info guide. Should their clinical care givers need help, contact info is provided.

We look forward to working with you and enhancing patient care with SecurAcath.

SecurAcath Info & Videos:



SecurAcath
Website



SecurAcath
Procedural Videos



SecurAcath Care and
Removal Handout



Patient Information
Guide



www.securacath.com
©2022 by Interrad Medical, Inc.
All rights reserved.

181 Cheshire Lane, Suite 100
Plymouth, MN 55441 USA
+1.763.225.6699

Please refer to instructions for use for
indications, contraindications, hazards,
warnings, cautions and directions for use.



Download the
SecurAcath® App



1329-041 Rev A